

PARKSVILLE BAY - AREA OF OPERATIONS BOUNDARY



STAY within the **AREA OF OPERATIONS**

Do not operate the Mobi-chair outside the above Area of Operations Boundary.

Risks include : High Winds, Strong Currents, Drowning, Exhaustion, Other Watercraft and Vessels. Mobi-chair is not intended for areas with current or strong waves.

The Mobi-chair is best propelled by pulling the tether-strap from the front. Additional support can come from pushing the rear handlebar.

If entering the water, pull the Mobi-chair in by the rear keeping the occupant informed when they will make contact with the water. Although the Mobi-chair floats, it is not a boat. The sitting position is stable and the occupant won't slip out of the Mobi-chair under normal conditions of use.

For safety reasons, a life jacket is required for non swimmers.





Although the Mobi-chair™ floats, it is not a boat. The sitting position is stable and you won't slip out of the Mobi-chair™ under normal conditions of use.

However, for safety reasons, a life jacket is required for non swimmers (this may also be required by law).

Mobi-chair™ is not intended for areas with current or strong waves.



ADVANTAGES



- > UV and sea water resistant
- > Mobile
- > Safe & robust
- > For both adults and children
- > Floating
- > Anticorrosion
- > Useful & ergonomic
- > Seamless transition from boardwalk-to beach-to water

Technical specificities

Folding frame	Aluminum frame & stainless fork and hardware, salt water resistant			
Mobi-chair dimensions®	Width	Length	Height	Weight
	107 cm 42"	152 cm 60"	127 cm 50"	30kg 66 lbs
Seat dimensions	Width	Depth	Height	Max load capacity
	46 cm 18"	43 cm 17"	48 cm 19"	136 kg 300 lbs
Adjustable backrest	3 positions: relaxation, seated, swimming			
Seat color	Blue			
Seat Fabric	UV Resistant - Water Resistant Ventilated - Non Allergenic			
Detachable Armrest	PVC Flip up - detachable floating armrest - low pressure			
Front wheel	30cm x 16cm 12" x 6-1/2» Rubber Pneumatic Tire*1 w/quick release			
Rear wheel	40cm x 19cm 16" x 7-1/2» Rubber Pneumatic Tire*2 w/quick release			

Adaptable & high quality wheelchair



- > **Easy mobility** through sand and other rough surfaces
- > **Safety harness**



- > **Rubber pneumatic** tires absorb the shock and discomfort caused by rugged terrain



- > **Maximum stability** with floating armrests and floating tires



- > **Handles integrated** into the armrests provide support
- > **Transportation and storage are convenient**

Quick installation & low maintenance

QUICK INSTALLATION


Assembly and disassembly of Mobi-chair™ without material and in 10 minutes maximum.

Completely foldable, Mobi-chair™ is easily transportable and fits perfectly in the trunk of a car.

EASY MAINTENANCE

Once a week, lubricate all connecting rods and safety screws.

After each use, rinse Mobi-chair® with fresh water to avoid corrosion and store it in a clean, dry place.

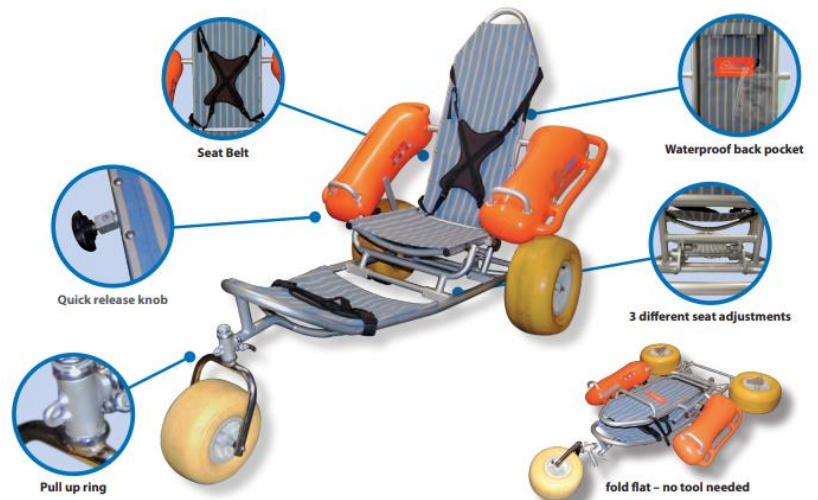

Multiple applications

SAND



OCEANS & SEAS



LAKES



SWIMMING POOLS



What are our complementary products ?

Mobi-mat®
Accessibility mat



Mobi-roller™
Mats dispenser



Mobi-com™
Advertising mat



Mobi-fence™
Sand fence

